

TrueDent™

True aesthetics made possible







Monolithic, Full-Color 3D Printed Dentures and Temporaries

TrueDent is an FDA cleared (Class II) resin developed for 3D printing of dentures and temporary crowns and bridges on the J5 DentaJet platform. It enables batch production of highly aesthetic, monolithic, multicolor dental appliances on a single mixed-part, high-capacity tray.

Monolithic Print

TrueDent dentures are a full-color, single continuous print of both denture base and teeth-no assembly required.

High Aesthetics

Proprietary PolyJet multi-material technology delivers natural looking gums and mimics tooth structure with optimal translucency. Print mixed batches of highly customized dentures and temporaries, matching patient demographic diversity on one tray.

Accurate print from design to fit

A high-fidelity match between design and print ensures repeatable, consistent production of accurate well-fitting dentures. This means less chair time—a win-win for patients, dentists, and labs.









True Digital Workflow

Step up denture and temporary crown and bridge production on the J5 DentaJet. Streamline your workflow to produce large batches of mixed, multi-color dental appliances in a single, unattended print.

Say goodbye to bonding

Print a denture prosthesis in a single monolithic structure. Cut down processing time and increase accuracy and consistency.

Multi-color printing

Simultaneous printing of polychromatic parts simplifies production process, increases efficiency, and expands your service offering. No need to change materials between runs.

Streamlined color workflow

Shade selection is exported from 3Shape Dental System via 3MF file format to GrabCAD Print and assigned automatically to the printed part.

Automated tray planning

GrabCAD Print software automatically adds support material, orients parts, and optimizes tray capacity saving time in the pre-production phase.

Software-enabled hardware

Software updates deliver improvements over time to the application's aesthetics and production capabilities of the printer, maximizing your return on investment.





Indications for use

Stratasys TrueDentTM is a light-curable resin indicated for the fabrication of dental appliances including removable full and partial dentures, denture bases, denture teeth, bridges, crowns, inlays, onlays, and veneers in dental laboratories. The material is an alternative to traditional heat-curable and auto polymerizing resins. Stratasys TrueDentTM is intended exclusively for professional dental work. Fabrication of dental appliances with Stratasys TrueDentTM requires a computer-aided design and manufacturing (CAD/CAM) system that includes the following components: digital dental files based on a digital impression, a Stratasys PolyJet 3D printer, and curing light equipment.

Stratasys J5 DentaJet TrueDent Specifications			
Materials	 TrueDent™ Cyan TrueDent™ Magenta TrueDent™ Yellow TrueDent™ White X TrueDent™ Clear X TrueDent™ Support 		
Container size	1.1 kg		
Available color presets	Gingiva 10 shades Teeth 16 shades (2-layer structure) 2 shades (3-layer structure)		
Batch size	Full tray ~45 models (Case dependent)		

Material Properties			
Property	Requirement	Value	Compliance
Ultimate flexural strength (MPa)	≥65 ≥50	≥85 ≥94	ISO 20795-1:2013 ISO 10477:2020
Flexural modulus (MPa)	≥2000	≥2300	ISO 20795-1:2013
Water Sorption (µg/mm3)	≤32 ≤40	≤32 ≤28	ISO 20795-1:2013 ISO 10477:2020
Solubility (µg/mm3)	≤1.1	≤1.1	ISO 20795-1:2013

Grow Your Lab with Stratasys

For more than 30 years Stratasys has led the global shift to additive manufacturing with 3D printing solutions for the aerospace, automotive, consumer products and healthcare industries. Through smart and connected 3D printers, multiple technologies, advanced materials, and a software ecosystem, Stratasys' solutions deliver a competitive advantage at every stage in the dental production process.

Stratasys Headquarters 7665 Commerce Way, Eden Prairie, MN 55344 +1 952 937-3000 (Intl)



ISO 13485:2016 certified

